Detergent reagent for Spincell 3 IVD

INTENDED USE

This product is designed to be used on a Spinreact 3-Part-Diff haematology analyser.

FUNCTIONS

With the functions of emulsification and catalysis, the Detergent emulsifies the fat, distributes the solidified dirt and transforms the protein into water-soluble amino-acids to maintain the cleanness of tubes and flow-cells. With effective humidification, it also discharges the bubbles from the entire flow system and ensures a normal working performance. The Detergent will not stain, corrupt the tubes or shorten instrument's service life.

INGREDIENTS

Chloride, sulfate, preservative solution, anticoagulant, buffer solution and surface active agent.

STORAGE & STABILITY

This product will be stable for 24 months when stored unopened at 2°C~40°C and away from sunlight. Once opened, this product should be consumed within 60 days and stored below 35°C

BLOOD SAMPLES

1. The blood samples should be venous and capillary specimens collected in EDTA-K₂ anticoagulant. Coagulated, hemolysis and lipidaemia samples cannot be measured.
2. Blood samples should be stored at 2°C~8°C.

OPERATIONAL PROCEDURE

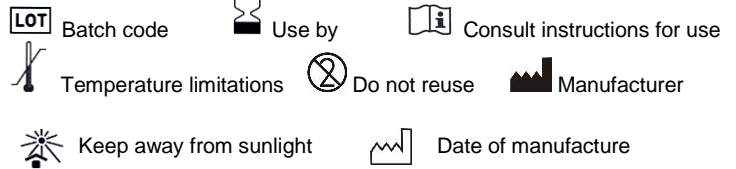
1. Connect detergent to tubing interface which is marked "DETERGENT" on the rear panel of the instrument.
2. Place a clean container filled with detergent beneath the probe when performing daily shutdown. The instrument will absorb detergent automatically when turning off, to clean apertures and tubing.
Read the operation manual carefully.

RESTRICTIONS, PRECAUTIONS & NOTICE

1. Use the reagent under the professional technician's guidance.
2. Make sure that the reagent matches with your analyzer. Any mismatch will lead to misleading results.
3. Avoid contacting with skin and eyes. In case of contact, rinse immediately with plenty of water and seek medical advice.
4. Avoid inhaling the evaporated reagent.
5. Tight up the container's cap to prevent from volatilization and contamination.
6. Once opened, dispose the remains over 60 days.
7. Avoid product freezing.

8. Concerning the sifting-out WBC identification, suggest performing manual coloration classification with microscope if the sample shows unusual results (such as WBC or RBC exorbitant and over-low).

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