Probe Cleaner for Spincell 3 and Spincell 5 Series IVD

INTENDED USE

Probe Cleaner is appropriate for SPINREACT automated hematology analyzers.

FUNCTIONS

Probe Cleaner is a reagent used to clean the instrument's probe or tubes. It can also be used for daily maintenance. The Probe Cleaner does not impact the instrument's use life, and will not stain or corrupt the tubes.

INGREDIENTS

Chloride, demineralized water, hypochlorite.

STORAGE & STABILITY

This product will be stable for 12 months when stored unopened at room temperature. Once opened, this product should be consumed within 60 days.

OPERATIONAL PROCEDURE

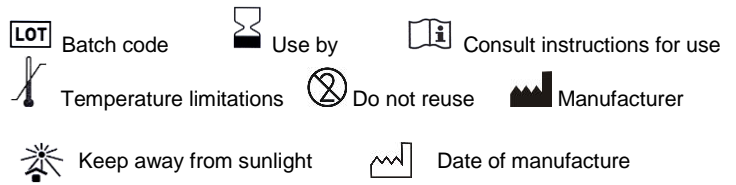
When serious clogging occurs or instrument has not been properly maintained for a time, the use of the probe cleaner is indicated.

1. Place a clean container with 2ml of probe cleaner reagent under the sample probe.
2. Select "Clean" in the operator interface (follow the operation manual). The instrument will aspirate probe cleaner automatically.

RESTRICTIONS, PRECAUTIONS & NOTICE

1. Use the reagent under professional technician's guidance.
2. Avoid contacting with skin and eyes. In case of contact, rinse immediately with plenty of water and seek medical advice.
3. Avoid inhaling the evaporated reagent.
4. Tight up the container's cap to prevent volatilization and contamination.
5. Once opened, dispose the remains over 60 days.
6. Avoid product freezing.

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